

Philips Monitor Waveforms to EMR



Intended Audience: Inpatient and ED physicians and APPs

Purpose: This tipsheet describes the process for saving tele strips from the Philips monitor to the Media tab in Epic.


Saving a Waveform Strip

If you are printing a waveform from the **central monitor**, you will save strips to Epic in the same way as they were printed before, but you will have the choice of where the printed report goes:

- **Paper** will print a paper copy to the designated printer
- **Electronic Document** will send a PDF copy to Epic saved in Chart Review under the Media tab
- **Paper and Electronic** will print a paper copy and send a PDF to the chart.

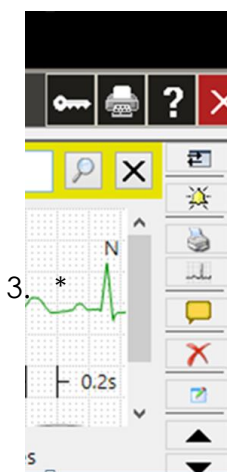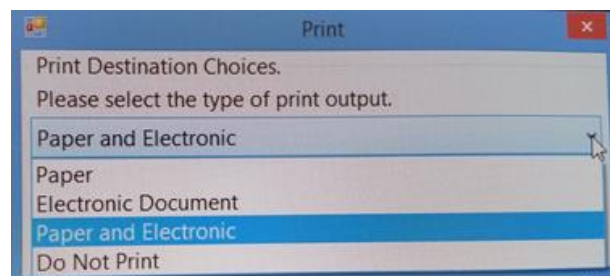


1. From the central station, start by selecting the portion of strip you want to print, then find the printer icon.
2. Choose your print destination and click print.

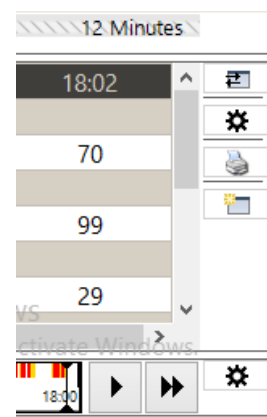

Finding Saved Waveforms

While we will no longer be printing routine rhythm strips, there are times where a provider may wish to send the tele strip to the patient's Epic chart for later reference. To find these saved strips:

1. Go to **Chart Review**
2. Click on the **Media** Tab
3. Check the filter for **Telemetry**

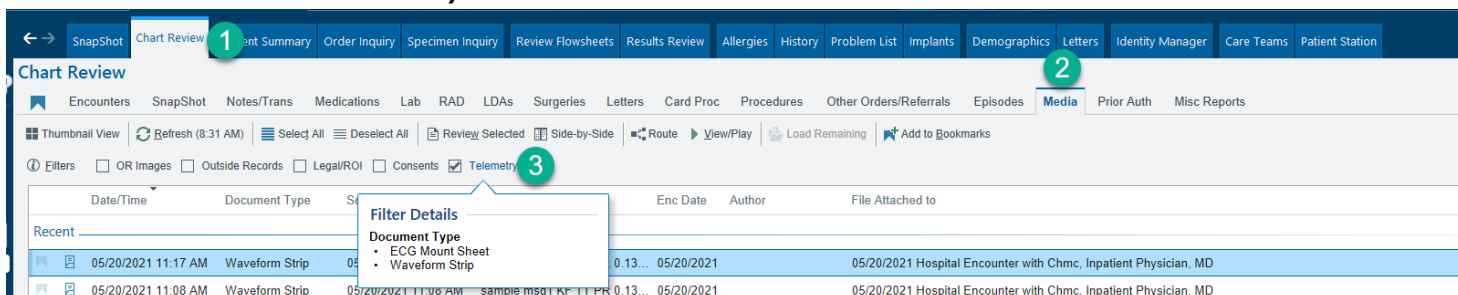

Tele wave strips in Epic will appear in a PDF format like this:

